


AudioConsole 4.5.0 Release Notes

We are excited to announce the release of AudioConsole 4.5.0, which brings significant improvements and new features designed to enhance your experience of the AudioConsole software. Here's what's new:

Modern Look and Improved user interface.


- **New Modern Look:** AudioConsole now boasts a sleek and modern interface, making it more user-friendly and visually appealing.
- **High Resolution Support:** The GUI has been optimized for high-resolution screens, ensuring crisp and clear visuals on all devices.
- **Minimalistic Icons:** We have introduced new minimalistic icons for a cleaner and more intuitive user experience.

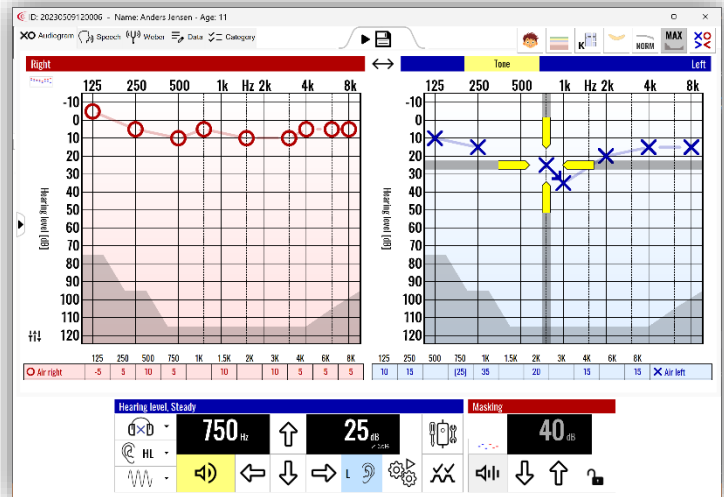

New Feature: Trio Autotest


We are proud to introduce the new "Trio Autotest", an automatic hearing screening test specifically designed to enhance the accuracy of results. The Trio Autotest consists of three phases:

1. **Initial Test:** Determines if the patient understands the test process.
2. **Basic Test:** Automatically suggests which frequencies should be re-tested during the screening.
3. **Re-test:** Conducted on the suggested frequencies to ensure accuracy. This phase guarantees more reliable results due to the thorough re-testing process.

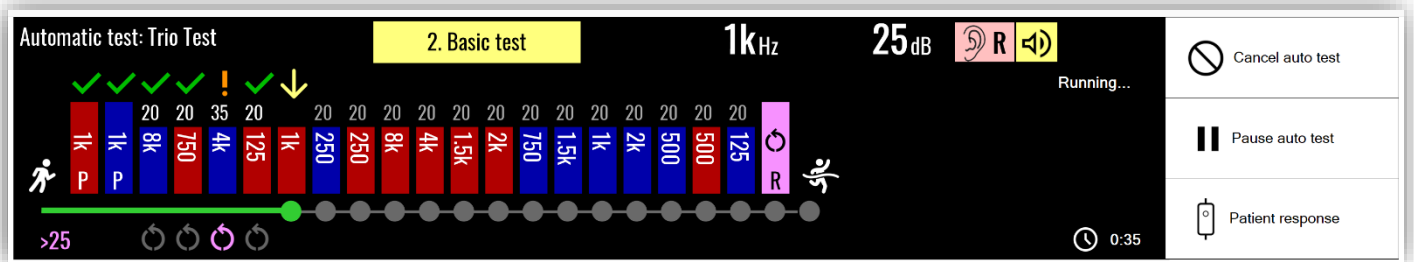

New Feature: Audiogram Explanation Layer

The new "Audiogram Explanation Layer" is designed to help operators explain the impact of hearing loss to children. Various audio sources are presented as illustrations based on frequency and intensity. For example, a bulldozer represents low, powerful sounds, while a bird singing represents higher, less intense sounds. This visualization aids in better understanding the audiogram results.

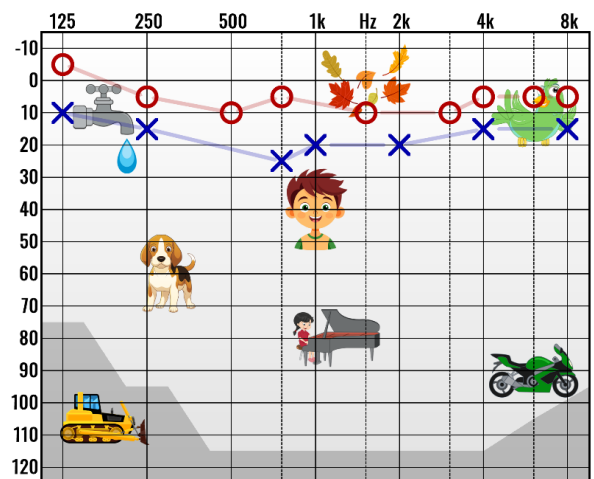

Upgrade Information

- **Free Upgrade:** Customers who have purchased our Oscilla A30, A50, or A60 models can upgrade to AudioConsole 4.5.0 free of charge.
- **Upgrade License Required:** Customers with legacy devices will need to purchase an upgrade license from their local Oscilla distributor.

Download AudioConsole 4.5.0: https://www.oscilla.dk/download/ac_setup_450.exe

We believe these updates will greatly improve your experience with AudioConsole and provide you with the tools you need to deliver accurate and understandable hearing assessments. Thank you for choosing AudioConsole.

For any questions or further assistance, please contact our support team or your local Oscilla distributor.